


MAY NATIONAL ARTHRITIS AWARENESS MONTH



May is recognized as National Arthritis Awareness Month, a disease that affects more than 60 million adults or about 1 in 4 US adults with doctor-diagnosed arthritis or another type of rheumatic disease. By the year 2040 it is estimated that 78.4 million adults (aged 18 years and older) will have doctor-diagnosed arthritis. It is the #1 cause of disability in the US accounting for more than 156 billion annually in lost wages and medical expenses.

Estimated and Projected Number of Adults with Doctor-Diagnosed Arthritis in the United States

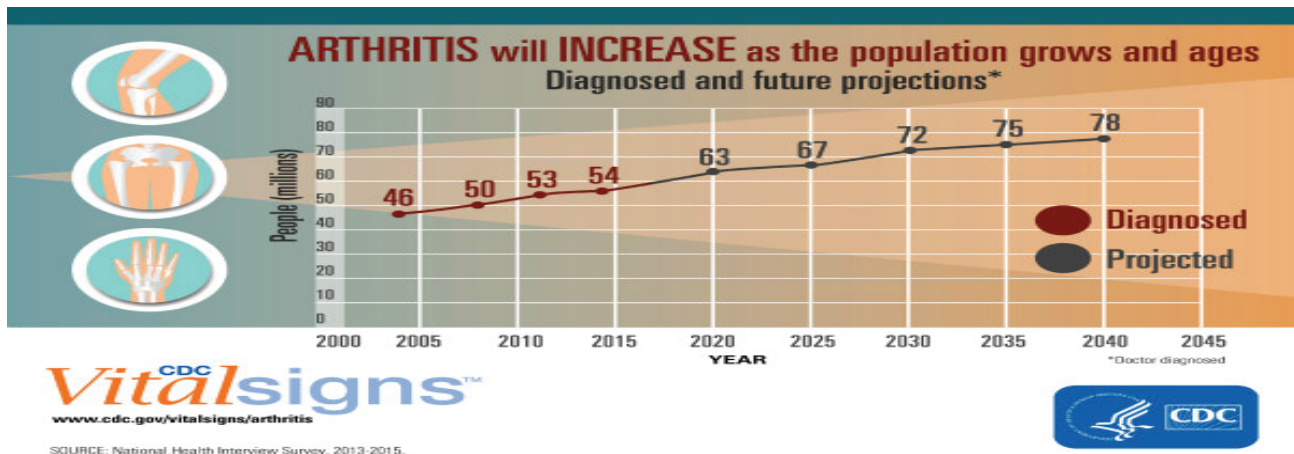

There are over 100 different types of arthritis or related conditions. Osteoarthritis (OA) is the most common form of arthritis affecting women more likely than men causing pain, swelling, stiffness, and decreased range of motion in joints. Anyone who has been injured or overused their joints, like athletes, military personnel, or people with physically demanding jobs, can also suffer with OA as they age. It can damage any joint but occurs more often in the hips, knees, lower back and neck, and other smaller joints. Any type of severe arthritis can cause chronic pain and the inability to do simple things, like walking, climbing stairs, or any daily activities.

RST-SANEXAS neoGEN®-Series Electric cell-Signaling Technology (EcST) is the #1 pain management and circulatory system to help manage patients who are suffering with acute or chronic pain associated with arthritis or other conditions. The device also helps maintain and increase range of motion and as well as other FDA cleared clinical indications to help manage your patient's outcomes in a safe, non-invasive, effective, and non-pharmaceutical way. Helping your patients to become more physically active will improve their overall health and well-being.

In this newsletter, we provide you with some guidelines for basic electrode placements on various areas of the body to help relieve pain, increase circulation, and improve range of motion and muscular reeducation. The neoGEN® device comes with 20 different custom programs that you can use to stimulate the recovery process with extremely high patient compliance outcomes and satisfaction. Patient outbound referrals can be substantially reduced.

Take care and thank you for all that you do!


KNEE PAIN

“Boxed”


“Diamond”

“Through”


“Boxed Through” or
“Linear” Approach

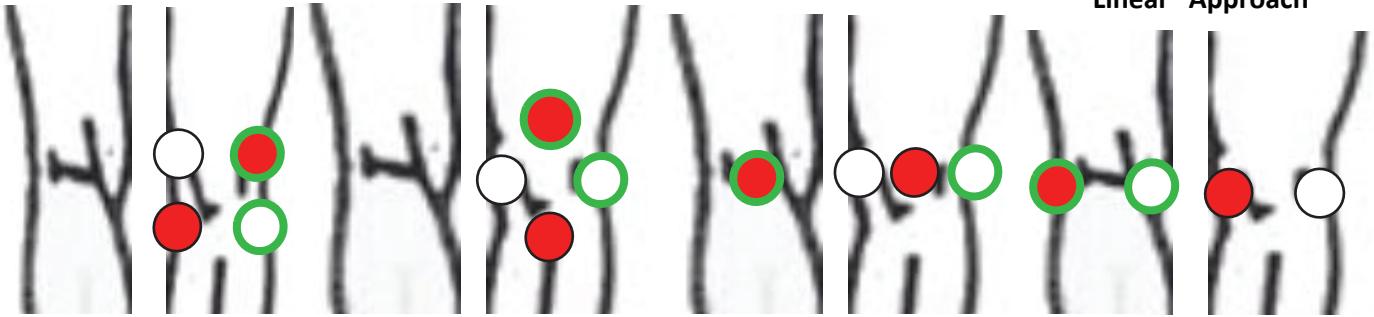


HIP PAIN Placement Variations

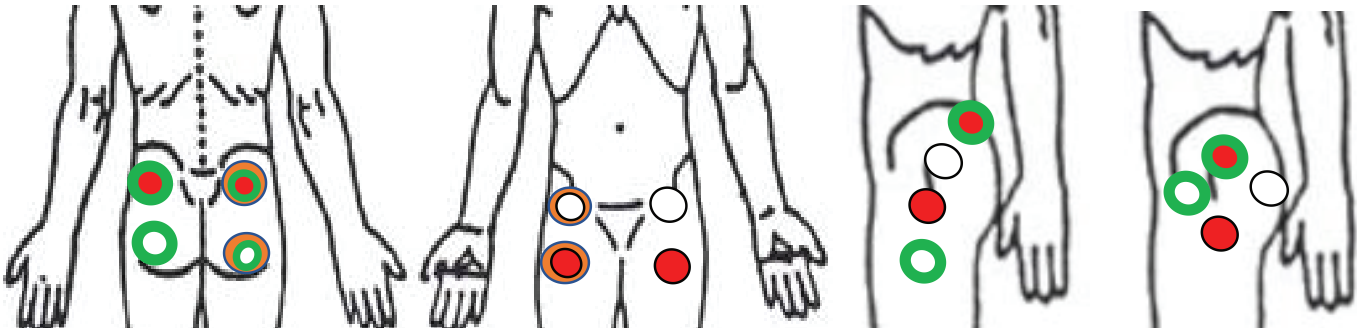

The above placements are the most common, but not only, hip targeting variations. The “Box” and “Diamond” approaches are used the most, and the “Linear” in more cases of radiating pain. All Placements can be moved, shifted, interlaced, etc. to target the area of pain.

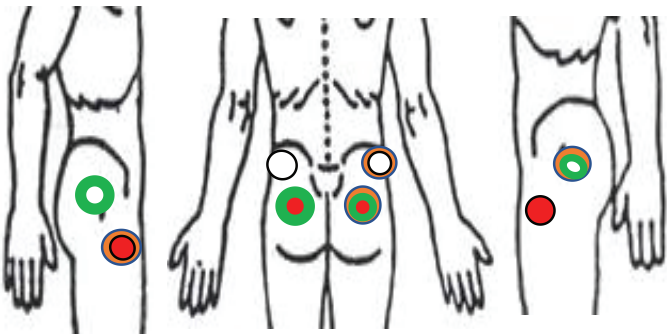

The above placement is one of many ways to target multiple areas by overlapping/crossing your channels.

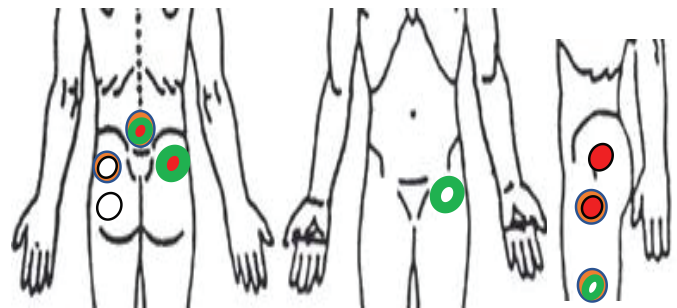

The placement to the right is one of many ways to target unilateral pain by overlapping/crossing your channels.


NECK/BACK -CERVICAL AREA

“Boxed”

“T”

“Diamond”

“Boxed Linear”

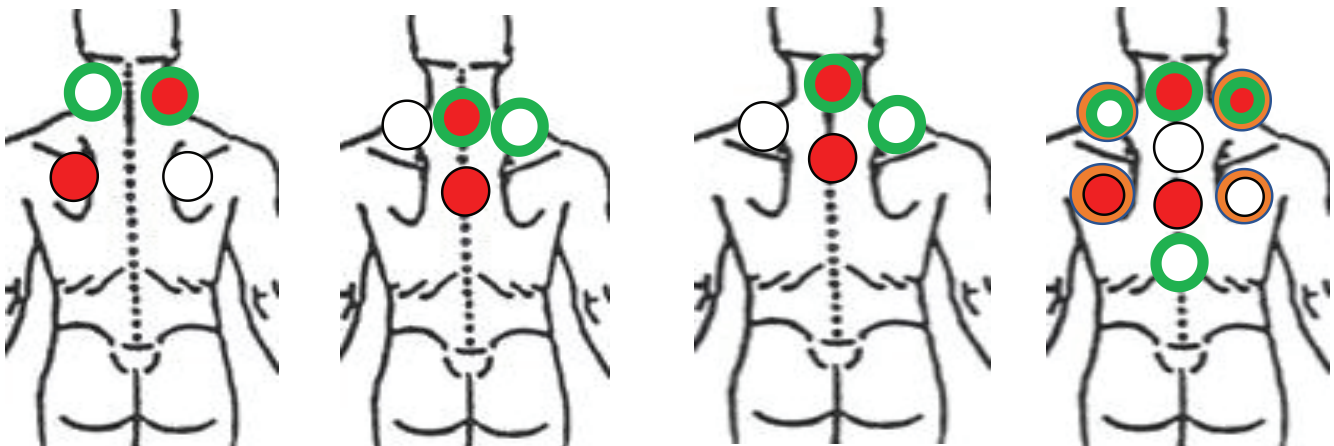



RST-SANEXAS CBD ANALGESIC BALM

fast-acting, long-lasting treatment **afterglow**



Shipping & Tax (if applicable)

1 oz cans:

1 for \$29 10 for \$250 (\$25 ea)
24 for \$480 (\$20 ea)

3 oz cans:

1 for \$49
5+ for \$45 each

This premium all-natural, patented technology uses polarized Hemp CBD and Arnica Montana, along with Turmeric, and a proprietary blend of essential oils for relief, comfort, and an overall treatment afterglow. The product is free of THC and has no psycho-active effect. The topical balm is absorbed into the skin giving fast-acting, long-lasting effects that works synergistically to help relieve discomfort from painful conditions.

Your patients can take home the non-greasy analgesic balm, apply a pea size amount to the source of their pain, and massage it in for 30-40 seconds to relieve discomfort in between treatments.

Call today to place your orders.

